


COVID-19 ANTIGEN SALIVA PEN TEST KIT


TEST RESULTS INTERPRETATION

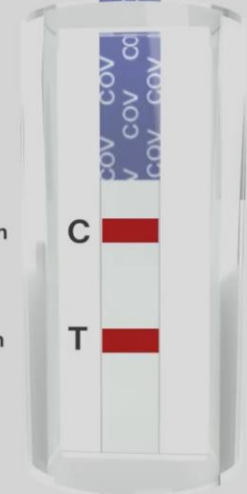

POSITIVE


Two Lines:
Line in Control Region (C) and any visible line in Test Region (T).

Note:
A very faint line in the Test Region (T) is also considered to be positive.

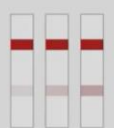

NEGATIVE

One Line:
Line in Control Region (C) and no apparent line in Test Region (T).

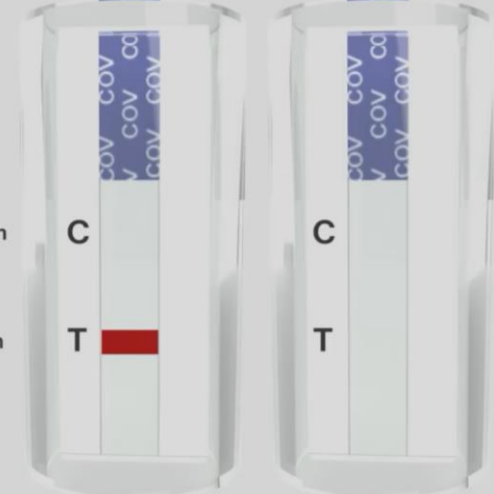

INVALID

No line in Control Region (C)

- Results from any test which has not produced a control band at the specified read time must be discarded
- Please review the procedure and repeat with a new test
- If the problem persists, discontinue using the kit immediately and contact your local distributor



Positive **Negative** **Invalid**